GCC-E / Enhanced

Clinical Centrifuge


Quick Start Guide

What's in the box:

- Quick Start Guide
- Centrifuge
- Power Cables (3 each)
- Rotor
- Wrench
- Door Key (small screwdriver)
- Sleeve Adapters (12 each)
- Riser Adapters (12 each)

UNPACKING THE UNIT

Please be sure to remove any packing materials used during initial shipment:

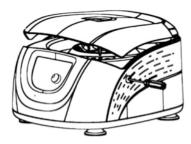

1) Open the lid: Use the included Door Key to press the button on the right side of the unit.



2) Remove foam pads at the hinge latch and from around the rotor.



LOADING INSTRUCTIONS

IMPORTANT: The rotor should always be loaded in a balanced manner to avoid personal injury and to ensure centrifuge longevity.

- When any tube is loaded, a tube of the same type, volume and weight must be added in the opposite rotor location.
- Never intentionally run the centrifuge in an unbalanced condition even if operation is not affected.

GCC-E / Enhanced

Clinical Centrifuge


Quick Start Guide

OPERATION

IMPORTANT: Plug unit into appropriate main supply based on the rated voltage.

- **Select** Press to choose the parameter you wish to modify.
- **Pulse** Press to accelerate and hold at the set speed. The centrifuge stops immediately once this button is released.
- **Door** Press to open the lid. This button is not enabled while the centrifuge is running.
- **Start / Stop** Press to start the centrifuge. Press to stop if centrifuge is already running.
- **Adjustment Knob** Rotate clockwise to increase parameter values. Rotate counterclockwise to decrease parameter values. Press to switch between speed and RCF.

CONTROL PANEL

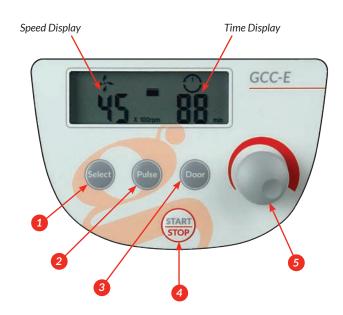

Rotor Type	Tube Type	Tubes Per Rotor	Dimension (ø×L mm)	Adapters Required	Maximum Speed (rpm)	Radius (cm)	Maximum RCF (×g)
CC-R1 (Standard Rotor)	15mL Conical	8	17×120	None	4500	11	2490
	1.5 - 5mL Vacuum	12	13×82	CC-AD1 CC-PLUG	4500	9.8	2218
	4 - 7mL Vacuum	12	13×106	CC-AD1	4500	11	2490
			16×75	CC-AD1 CC-PLUG		9.8	2218
	8.5 - 10mL Vacuum	12	16×107	CC-AD1	4500	11	2490
	2.7 - 3mL (EU) Collection	12	11×66	CC-AD1 CC-PLUG	4500	9.8	2218
	7.5 - 8.2mL (EU) Collection	12	15×92	CC-AD1	4500	11	2490

For the full User Manual, please visit www.globescientific.com



Customer Service, Sales and Technical Support:

Phone: 1 (800) 394-4562 • 1 (201) 599-1400

1 (201) 599-1406 • E-mail: mail@globescientific.com