
GCC-S / Standard Clinical Centrifuge Quick Start Guide

What's in the box:

- Quick Start Guide
- Centrifuge
- Power Cable
- Rotor
- Wrench
- Sleeve Adapters (12 each)
- Riser Adapters (12 each)

UNPACKING THE UNIT

Please be sure to remove any packing materials used during initial shipment:

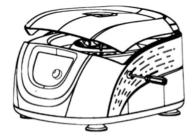

1) Press the button on the right side to open the lid.



2) Remove foam pads at the hinge latch and from around the rotor.



LOADING INSTRUCTIONS

IMPORTANT: The rotor should always be loaded in a balanced manner to avoid personal injury and to ensure centrifuge longevity.

- When any tube is loaded, a tube of the same type, volume and weight must be added in the opposite rotor location.
- Never intentionally run the centrifuge in an unbalanced condition even if operation is not affected.



GCC-S / Standard Clinical Centrifuge Quick Start Guide

OPERATION

Starting the Unit:

IMPORTANT: Plug unit into appropriate main supply based on the rated voltage.


- Press the power button and the display screen will light up.
- Press the speed buttons to adjust to the desired setting.
- Press the time buttons to adjust to the desired time setting. For set times above 60 seconds, the "min" (minute) light will be lit. Under 60 seconds, the "sec" (second) light will be lit.
- Press the "RUN/STOP" button to start the centrifuge.

Stopping the Unit:

There are three conditions under which the centrifuge will stop running:

- When the set time runs out.
- Pressing the "RUN/STOP" button before the set time runs out.
- Opening the lid.

CONTROL PANEL

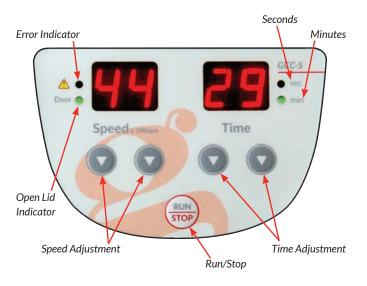

Rotor Type	Tube Type	Tubes Per Rotor	Dimension (ø×L mm)	Adapters Required	Maximum Speed (rpm)	Radius (cm)	Maximum RCF (×g)
CC-R1 (Standard Rotor)	15mL Conical	8	17×120	None	4500	11	2490
	1.5 - 5mL Vacuum	12	13×82	CC-AD1 CC-PLUG	4500	9.8	2218
	4 - 7mL Vacuum	12	13×106	CC-AD1	4500	11	2490
			16×75	CC-AD1 CC-PLUG		9.8	2218
	8.5 - 10mL Vacuum	12	16×107	CC-AD1	4500	11	2490
	2.7 - 3mL (EU) Collection	12	11×66	CC-AD1 CC-PLUG	4500	9.8	2218
	7.5 - 8.2mL (EU) Collection	12	15×92	CC-AD1	4500	11	2490

For the full User Manual, please visit www.globescientific.com



Customer Service, Sales and Technical Support: Phone: 1 (800) 394-4562 • 1 (201) 599-1400 Fax: 1 (201) 599-1406 • E-mail: mail@globescientific.com