

GCC-S / Standard Clinical Centrifuge


Quick Start Guide

What's in the box:

- Quick Start Guide
- Centrifuge
- Power Cable
- Rotor
- Wrench
- Sleeve Adapters (12 each)
- Riser Adapters (12 each)

UNPACKING THE UNIT

Please be sure to remove any packing materials used during initial shipment:

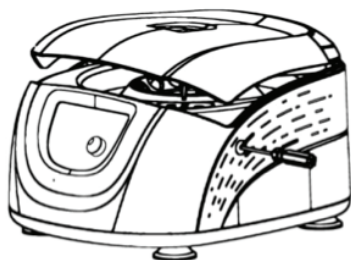

1) Press the button on the right side to open the lid.



2) Remove foam pads at the hinge latch and from around the rotor.



LOADING INSTRUCTIONS

IMPORTANT: The rotor should always be loaded in a balanced manner to avoid personal injury and to ensure centrifuge longevity.

- When any tube is loaded, a tube of the same type, volume and weight must be added in the opposite rotor location.
- Never intentionally run the centrifuge in an unbalanced condition even if operation is not affected.

Continued...

GCC-S / Standard Clinical Centrifuge

Quick Start Guide

OPERATION

Starting the Unit:

IMPORTANT: Plug unit into appropriate main supply based on the rated voltage.


- Press the power button and the display screen will light up.
- Press the select button to choose the parameter you wish to modify. Then...
- Rotate the adjustment knob clockwise to increase parameter values or counterclockwise to decrease parameter values. Press to switch between speed and RCF.
- Press the "RUN/STOP" button to start the centrifuge.

Stopping the Unit:

There are three conditions under which the centrifuge will stop running:

- When the set time runs out.
- Pressing the "RUN/STOP" button before the set time runs out.
- Opening the lid.

CONTROL PANEL

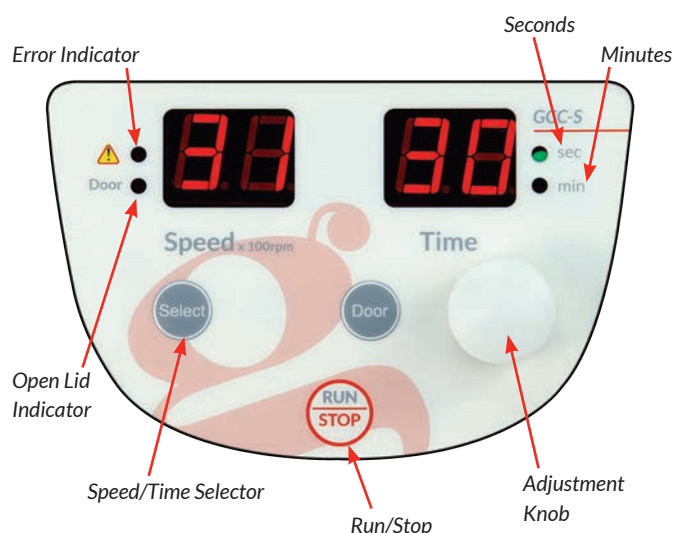

Rotor Type	Tube Type	Tubes Per Rotor	Dimension (ø×L mm)	Adapters Required	Maximum Speed (rpm)	Radius (cm)	Maximum RCF (×g)
GCC-R1 (Standard Rotor)	15mL Conical	8	17×120	None	4500	11	2490
	1.5 - 5mL Vacuum	12	13×82	CC-AD1 CC-PLUG	4500	9.8	2218
	4 - 7mL Vacuum	12	13×106	CC-AD1	4500	11	2490
			16×75	CC-AD1 CC-PLUG		9.8	2218
	8.5 - 10mL Vacuum	12	16×107	CC-AD1	4500	11	2490
	2.7 - 3mL (EU) Collection	12	11×66	CC-AD1 CC-PLUG	4500	9.8	2218
	7.5 - 8.2mL (EU) Collection	12	15×92	CC-AD1	4500	11	2490

For the full User Manual, please visit www.globescientific.com



Customer Service, Sales and Technical Support:

Phone: 1 (800) 394-4562 • 1 (201) 599-1400

Fax: 1 (201) 599-1406 • E-mail: mail@globescientific.com